	Größe 1 / Size 1	Größe 2 / Size 2	Größe 3 / Size 3
kleinster Handgelenk-Umfang / smallest wrist circumference	12 cm	15 cm	18 cm
größter Handgelenk-Umfang / largest wrist circumference	15 cm	18 cm	21 cm
Riemenbreite / strap width	5 cm	5 cm	5 cm
Art.-Nr. D... / Art.-No. E...	4 1 800=1	4 1 800=2	4 1 800=3

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Safety Notes

To reduce risk, please ensure that the instructions have been read and fully understood before use. The assembly instructions must be strictly adhered to.

Application

Starting with the wrist strap. This should fit snugly round the wrist with a fingers width play. The two vertical side straps should then be positioned at a slight angle downwards so that they do not chaff on the joints.

The position of the camlocks can now be marked on the armrest.
Using the pop rivets provided, the snap locks can now be mounted.

After the camlocks have been fitted to the armrests, fit the wrist strap to the clients wrist.
Thread the side straps through the camlocks and adjust the play in the elastic side straps until the correct arm position has been achieved.

Pull the harness straps downwards until the arms are positioned correctly and lock the camlocks, which are fastened to the loops.

The straps may be shortened between the seams. To prevent fraying the cut ends should be carefully singed with a lighter or similar.

Attention

Wrist Straps should be used as a restraint and positioning aid for the arms.
They should be used in conjunction with a fixed armrest.
The strengthened version should not be used by strong thrust spasticity or spasms.

Care and Maintenance

Use a soft brush to remove stains.
Washable at 30° with a mild detergent. Do not use fabric conditioner.

EC Declaration of Conformity

according to annex VII of the Council Directive 93/42/EEC concerning medical devices
We, McLean REHAtechnik GmbH declare under our sole responsibility that the products described in this instruction sheet meet the provisions of the Council Directive 93/42/EEC concerning medical devices which apply to them.