Technische Daten / Technical information

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Safety Notes

To reduce risk, please ensure that the instructions have been read and fully understood before use. The assembly instructions must be strictly adhered to.

Application

To achieve an ideal foot positioning, the footplates may be installed on wheelchairs, standing frames, bicycles and therapeutic chairs. They can be mounted onto a stable footrest or for example to a bicycle pedal. A range of harnesses with buckles, locks or hook-and-loop fasteners. Padded, sleeves, elasticated straps and Neoprene Harnesses are also available.

Specifications

The Footshells can be provided in two models: aluminium and vacuum moulded plastic. The plastic version is supplied in four sizes. The Aluminium Footshells are available in three sizes. The light thermoformed plastic Footshells are very durable and offer the advantage of rounded edges. Four belt slots are pre-cut. The rounded shape provides good protection against injuries. Due to its contour the Footshell offers a very sturdy form. The Footshells are fitted with three mounting points. The black powder-coated Aluminium Footshells are equipped with slots for easy fitting of straps. Three countersunk holes are pre bored for easy mounting. Please note: Due to the different mounting surfaces and assembly options screws and other mounting parts are not included in the delivery. If you require such materials, please contact us. We can offer you a variety of mounting options.

Assembly and Fitting

Through versatile use it is often necessary to provide an individual solution. Of great importance is that no screw heads are sticking up in the foot area, as this could lead to injuries, especially if no shoes are worn.

The Footshells should be fastened with at least three screws onto the Foot plate. For this use three M5 screws. Regarding the mounting of the plastic Footshells it is of great importance that a correct washer be placed between the screw and the countersunk assembly point, to avoid the screws ripping out.

To prevent injuries when using the Aluminium model, Trimband Art. No. E 1 2 012 could be used.

Care and Maintenance

Both Aluminium Shell models can be washed or brushed with hot water and a mild detergent. Do not use any solvent or a similar agent, as that could lead to damage.

EU Declaration of Conformity

We, McLean REHatechnik GmbH, declare under our sole responsibility that the products described in these operating instructions meet the basic requirements of the regulation (EU) 2017/745 (MDR).

Version: 12.12.2023

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All rights reserved.