
Technische Daten / Technical information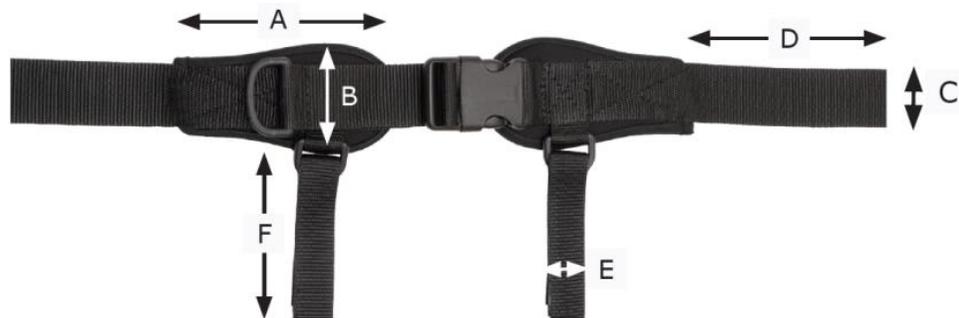

Art Nr. / Art. No.	A	B	C	D	E	F
D 1 5 440=25 D 1 5 441=25	100 mm	50 mm	25 mm	750 mm	20 mm	650 mm
D 1 5 440=30 D 1 5 441=30	125 mm	60 mm	30 mm	750 mm	25 mm	650 mm
D 1 5 440=40 D 1 5 441=40	150 mm	70 mm	40 mm	750 mm	25 mm	650 mm
D 1 5 440=50 D 1 5 441=50	170 mm	85 mm	50 mm	750 mm	25 mm	650 mm

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Safety Notes

To reduce risk, please ensure that the instructions have been read and fully understood before use. The assembly instructions must be strictly adhered to. Not suitable as restraint for transport in vehicles.

Application

The Lap Belt is used to aid pelvic positioning. Our Lap Belts are available in different sizes (25 - 50 mm width).

For children, the 25 mm up to 40 mm is recommended and for adults the 50 mm.

The Lap Belts are delivered with plastic fastenings but are also available with metal fastenings for greater support.

Assembly and Fitting

The Lap Belt should be fixed so that the straps sit at an angle of approximately 45°.

With 4-Point-Belts the additional straps (25 mm) should run straight down if possible.

The Lap Belt needs to be fitted tightly to the pelvis (max. 2 cm space), otherwise there is danger of sliding under the Lap Belt!

Mounting

Lap Belts should be attached to a firm backrest using the enclosed fastenings.

These are then either screwed onto the tube or fixed using the multiscrews enclosed.

Care and Maintenance

The Lap Belt is easy to care for. It is washable at 30°.

Use mild detergent. Do not spin dry. Do not tumble.

EU Declaration of Conformity

We, McLean REHAtechnik GmbH, declare under our sole responsibility that the products described in these operating instructions meet the basic requirements of the regulation (EU) 2017/745 (MDR).

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All rights reserved.