
## Technische Daten / Technical information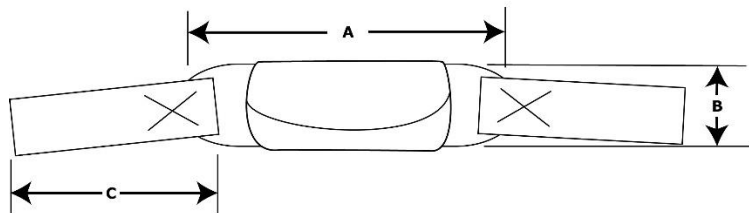

	A	B	C
Größe/ Size 1	22,5 cm (20-27)	6 cm	50 cm
Größe/ Size 2	30 cm (27-35)	6,5 cm	60 cm
Größe/ Size 3	40 cm (37-45)	8,5 cm	70 cm

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## Safety Notes

**To reduce risk, please ensure that the instructions have been read and fully understood before use. The assembly instructions must be strictly adhered to. Not suitable as restraint for transport in vehicles.**

## Application

Our Dynamic Torso Belts are manufactured of elastic Neoprene and are ideal for providing trunk support. The neoprene stretches to allow a degree of movement rather than restricting all freedom of movement. The fastenings can be adjusted to suit each users requirements.

## Assembly and Fitting

The Dynamic Torso Belt should be fitted in the horizontal plane. The tension should then be adjusted to the required level.

## Mounting

The Dynamic Torso Belt can be fixed to a seating system or a wheelchair with a firm backrest. Use the enclosed straps and fastenings. The straps are then either fixed to the frame by using existing holes in the backrest tubes or into the seating system frame. For attachment to wheelchair-frames (tubes) order tube loops (E 6 4 600).

## Care and Maintenance

The Dynamic Torso Belt is easy to care for. It is washable at 30°. Use mild detergent. Do not spin dry. Do not tumble dry.

## EU Declaration of Conformity

We, McLean REHatechnik GmbH, declare under our sole responsibility that the products described in these operating instructions meet the basic requirements of the regulation (EU) 2017/745 (MDR).

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All rights reserved.