
Technische Daten / Technical information

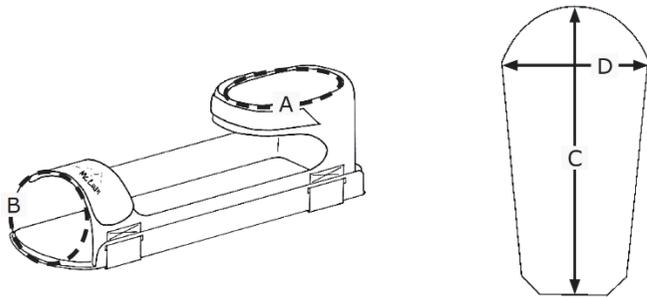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Safety Notes

To reduce risk, please ensure that the instructions have been read and fully understood before use. The assembly instructions must be strictly adhered to.

Application

To ensure correct foot positioning of the feet, the Flexi Foot Shells may be fitted to wheelchairs, standing frames, movement trainers and therapeutic chairs. They can be mounted onto most stable footrests, foot bars, foot plates, pedals and similar.

The elastic material ensures that movement is not completely restricted.

Individual adjustment can be carried out to cover a whole range of measurements and to accommodate different disabilities.

Assembly and Fitting

This assembly should only be performed by a qualified technician.

Firstly remove the sole pad from the foot shell. The sole pad is secured using hook and loop fasteners.

In the Foot Shells there are 3 pre-bored countersunk holes. Using the 3 countersunk M6 screws supplied, the foot shells can now be secured in the selected position. Check beforehand that the hook-and-loop fastener under the shell has been safely secured.

The sole pad may now be re-fitted. Open the fastening straps to position the foot in the Flexi Foot Shell and fasten it again.

Make sure that the straps are not too tight and that the foot is positioned correctly.

Due to the various mounting surfaces and fitting options no fastening material is included.

Should you require this material, please contact us. We can offer you a whole range of mounting op-

Attention

The Flexi Foot Shells should only be fitted onto stable footrests, foot bars, foot plates, pedals and similar. The maximum weight limit on the footshell must not exceed the maximum weight limit recommended by the manufacturer of the wheelbase to be used. (Please consult manufacturers specifications or handbook. The maximum weight limit may vary depending on the manufacturer).

In order to prevent injury when seated, the Flexi Foot Shells should only be used in combination with a pelvic restraint. When transferring, always check that the foot straps have been released. Soft shoes such as slippers, moccasin shoes and -boots may affect the correct fit of the Flexi Foot Shells.

Care and Maintenance

Light staining can generally be brushed out with a soft brush.

Neopren pads are machine washable at 30°. Do not use fabric conditioner.

Do not spin or tumble dry.

The powder coated aluminium foot shell may be cleaned with a damp cloth.

EU Declaration of Conformity

We, McLean REHAtechnik GmbH, declare under our sole responsibility that the products described in these operating instructions meet the basic requirements of the regulation (EU) 2017/745 (MDR).

Version: 21.08.2020

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All rights reserved.