Technische Daten / Technical information

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All dimensions are in cm. Due to the flexibility of the harnesses, the measurements given are average. Variations in the dimensions may occur!

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Safety Notes

To reduce risk, please ensure that the instructions have been read and fully understood before use. The assembly instructions must be strictly adhered to.

Application

Starting with the Ankle Strap. This should fit snugly round the ankle with a fingers width play. The two vertical side straps should then be positioned at a slight angle downwards so that they do not chaff on the ankle joints or the side of the foot.
The correct position of the metal guide bars can now be marked on the footrest.
Using the screws provided these can now be fixed to the footrest.
Ensure that you have pushed the belt loops with the slide lock over the metal guide loop before screwing them to the footrest.
After fastening the metal guide bars with the screws and nuts provided, fit the ankle straps.thread the two belts (fixed to the sides) through the cam locks. Pull the belts downwards until feet are positioned correctly. Then secure the straps with the snap buckles fixed to the loops.

Attention

Ankle Straps should be used as a restraint and positioning aid for the feet.
They should be used in conjunction with a fixed footrest/footboard.
The Ankle Straps should always be used together with a pelvic restraint to prevent injury.
The strengthened version should not be used by strong thrust spasticity or spasms.

Care and Maintenance

Use a soft brush to remove stains.
Washable at 30° with a mild detergent. Do not use fabric conditioner.

EU Declaration of Conformity

We, McLean REHatechnik GmbH, declare under our sole responsibility that the products described in these operating instructions meet the basic requirements of the regulation (EU) 2017/745 (MDR).

Version: 17.08.2020

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All rights reserved.