
Technische Daten / Technical information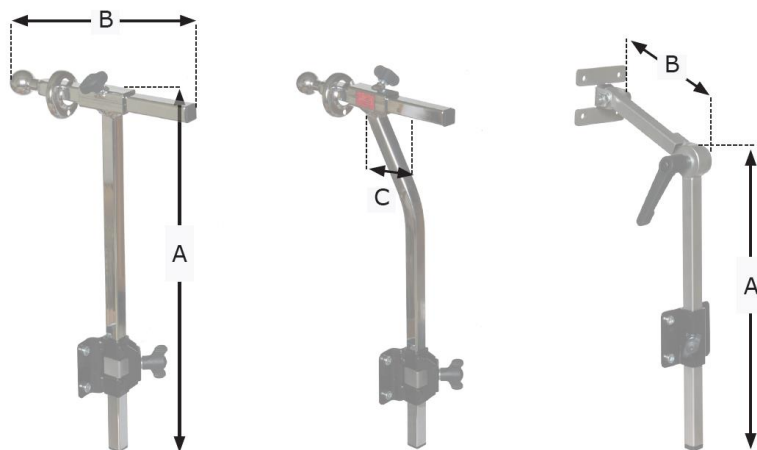

	gerade/ straight	gekröpft/ offset	Konugelenk/ coned joint	Konugelenk/ cond joint
	D 1 3 010	D 1 3 011	D 1 3 013=1	D 1 3 013=2
A=	40,0 cm	40,0 cm	32,0 cm	32,0 cm
B=	20,0 cm	20,0 cm	13,0 cm	16,0 cm
C=		6,0 cm		

A= Höhe/ Height, B= Breite/ Width, C= Kröpfung/ Offset

Lieferumfang/ Delivery includes

D 1 3 010/ D 1 3 011:

4x M6 Flachmutter/ Flatnut
4x M6 x 12 Innensechskantschraube/ Inner Allen Screw
4x U-Scheibe für M6/ Washer for M6
1 Unihalter D 1 4 032/ Unibacket
1 Befestigungsring/ Mounting Ring D 1 3 001

D 1 3 013=1

6x M6 Flachmutter/ Flatnut
6x M6 x 12 Innensechskantschraube/ Inner Allen Screw
6x U-Scheibe für M6/ Washer for M6
1 Unihalter/ Unibacket D 1 4 038
1 Anschraubplatte, mini/ Mounting Plate, mini D 1 3 003
1x M6 x 40 Innensechskantschraube/ Inner Allen Screw

D 1 3 013=2

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Safety Notes

To reduce risk, please ensure that the instructions have been read and fully understood before use. The assembly instructions must be strictly adhered to.

Application

The Headrest Hardware is used to mount the Headrest to seating systems with hard backrests or similar. It offers a range of vertical and horizontal adjustment.

Assembly and Fitting

The Headrest Hardware can be mounted using the universal clamp supplied with this kit. The clamp is fitted using 4 screws M 6 x 12 by boring 4 holes, 6,2 mm. The vertical tube may now be clamped into position. The horizontal tube with the ball & mounting ring may now be fitted with the three screws to the Headrest. Finally the Hardware and Headrest should be adjusted to the user.

Attention

All the screws and nuts supplied with this kit should only be used once.

Care and Maintenance

Clean with mild detergent and water.
Care only with special chrome polish.

EU Declaration of Conformity

We, McLean REHatechnik GmbH, declare under our sole responsibility that the products described in these operating instructions meet the basic requirements of the regulation (EU) 2017/745 (MDR).

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All rights reserved.