
Technische Daten / Technical information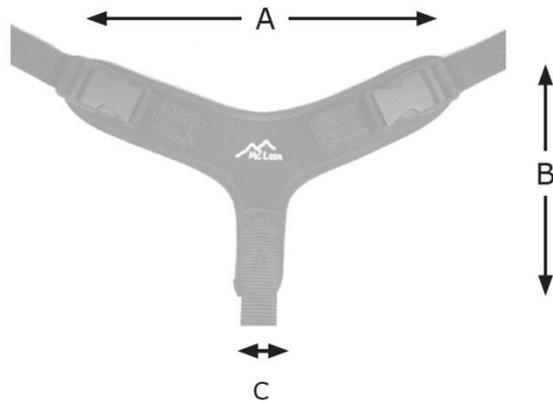

| Art. Nr/ Art. No. | D 15 501=1 | D 15 501=2 | D 15 501=3 |
|----------------------|---------------|---------------|---------------|
| | Größe/ Size 1 | Größe/ Size 2 | Größe/ Size 3 |
| A= | 22,0 cm | 28,0 cm | 34,0 cm |
| B= | 15,0 cm | 17,0 cm | 21,0 cm |
| C= | 3,5 cm | 4,0 cm | 4,5 cm |

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Safety Notes

To reduce risk, please ensure that the instructions have been read and fully understood before use. The assembly instructions must be strictly adhered to.

Application

Our Pelvic Harness is manufactured using elastic Neoprene and are used principally to assist pelvic positioning (e.g. posterior pelvic tilt). The Neoprene stretches to allow a degree of movement rather than restricting all freedom of movement. The fastenings can be adjusted to suit each user's requirements. Our Pelvic Harnesses are available in different sizes and two types.

Please ensure that the Pelvic Harness lies flat on the seat surface and without any creases. Only then should the user be positioned into the seating system.

Our Pelvic Harnesses have two fastening straps at each end. These straps are to be fed upwards between the legs and diagonally upwards and towards the back.

Assembly and Fitting

Our Pelvic Harnesses are most suited to seating systems and wheelchairs with hard backrests or seats. They can be fastened using the enclosed snap locks, which can either be screwed on or affixed to the Aluminium Frame (E 1 2 100) using the enclosed pop rivets. The exact positioning of the snap locks should be agreed with the therapist.

When fitting the Pelvic Harness to standard wheelchairs without a seating shell, use the appropriate tube loops (E 1 5 990=25/30/40/50).

Attention

This pelvic harness is a positioning aid. It is not a safety belt.

Care and Maintenance

Our Pelvic Harnesses are easy to care for and are machine washable at 30°. **Use a mild detergent, do not spin dry. Do not tumble dry.**

EU Declaration of Conformity

We, McLean REHatechnik GmbH, declare under our sole responsibility that the products described in these operating instructions meet the basic requirements of the regulation (EU) 2017/745 (MDR).

Version: 17.08.2020

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All rights reserved.